

LipoLASER Plus Evolution II



NOW FEATURING MORE POWER -160 mW PER LASER DIODE. MORE LASERS -128 LASERS

Our All-New LipoLASER Plus Evolution II has just arrived and is the newest addition to our professional lipo Laser lineup. It represents the state of the art and has more features and benefits that any system available anywhere. Based on our long term experience with professional therapy lasers, we have designed and introduced a system that lets you easily and conveniently perform non-invasive Lipo Laser on your clients as well as perform numerous beauty applications. It has 12 large laser pads, each with 10 160mW laser diodes and 4 small laser pads, each with 2 160mW laser diodes. With all this flexibility and power, you can target more areas at once greatly cutting down the time spent with clients and maximize revenue. It is now available at an incredible introductory price as long as you agree to participate in our clinical evaluation programs.

The LipoLaser Plus Evolution II is the most advanced Lipo Laser available. It features a host of state of the art features including a touch screen display, user controlable power of each pad, continuous wave or two pulsed frequencies, 16 laser pads with 160mW high powered laser diodes and so many more features.




Loose Weight at the Speed of Light
The Laser Therapy Lipolaser Plus slimming system is specially designed for non-invasive body contouring.



The LIPOLASER PLUS emulsifies adipose tissue while being applied externally. The Laser Therapy Lipolaser Plus slimming system is a low level laser therapy device specifically designed to treat body fat tissues and to help shape and tone the skin without the need of surgery. DO NOT CONFUSE THIS LIPO THERAPY SYSTEM WITH THE INVASIVE LIPO SYSTEMS THAT PENETRATE THE SKIN WITH A NEEDLE OR DIODE. THE LIPOLASER PLUS DOES NOT PHYSICALLY PENETRATE THE SKIN & IS PAINLESS & COMPLETELY SAFE. ONLY THE COHERENT LASER LIGHT PENETRATES THE SKIN.

The American Society of Lasers in Medicine and Surgery in 2008 presented a study of low level laser therapy for slimming which subsequently resulted in the first FDA clearance and approval for this type of laser.

Within weeks low-level laser helped patients lose an average of 5-6 inches from their waists, hips and thighs. The Lipolaser Plus slimming system works by emitting cold laser energy into the body tissues. By absorbing energy, fat in the cells are broken down and then channeled out of the body. The lipo laser virtually stimulates the break down and removal of body fat without affecting the surrounding skin and tissues.

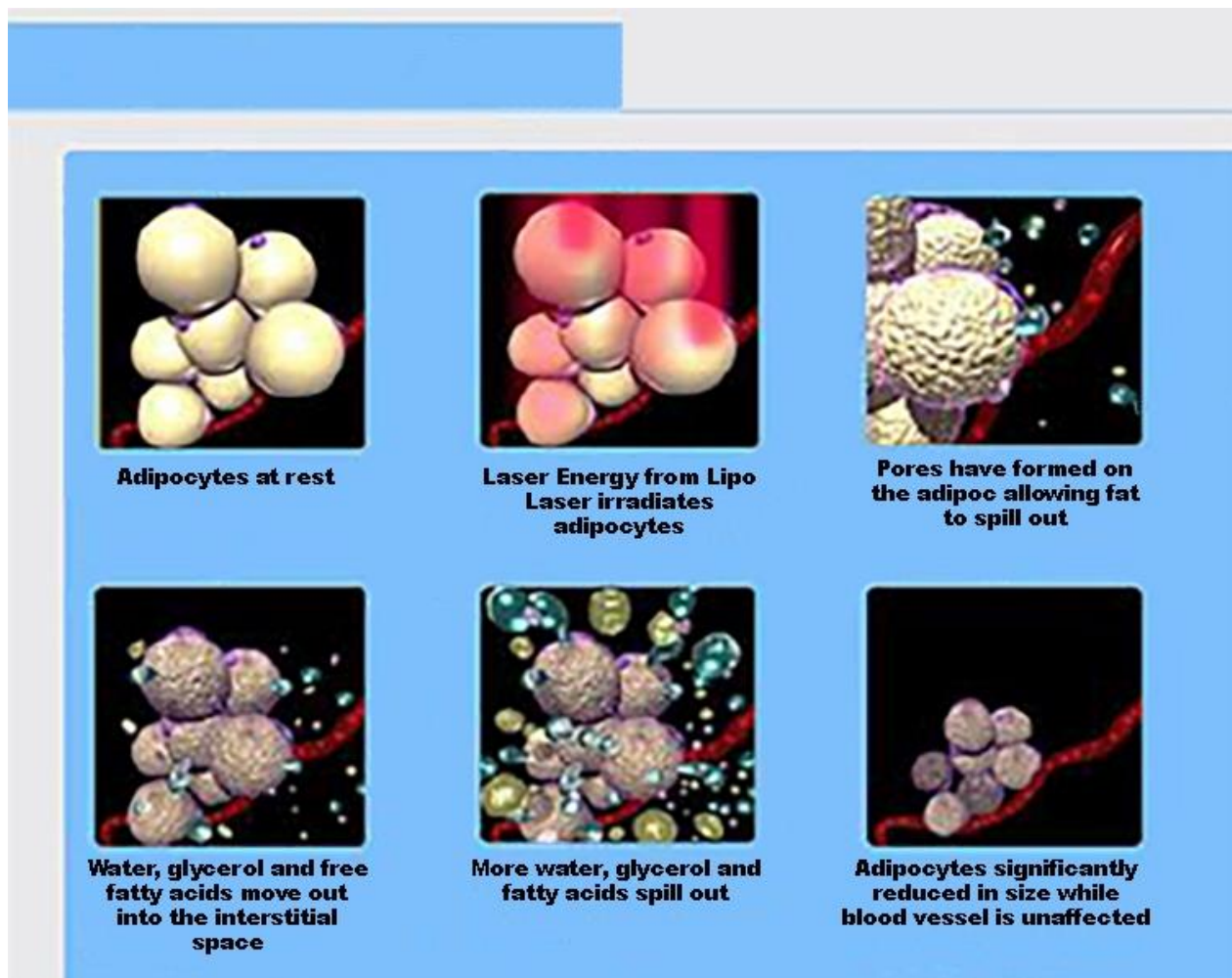


Lipo Laser Pads



Lipo Laser Pads placed on the body during treatment



Lipo Laser Pad Holder

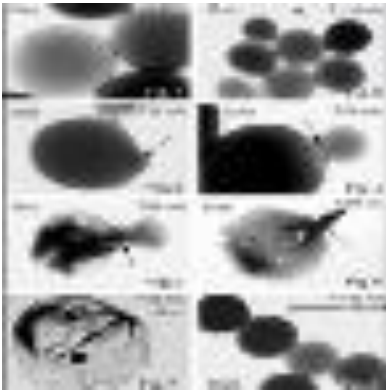


Here is an example how the lipo laser procedure works:


The diagram on the left shows time released photography of one fat cell and its contents being emptied. Please note the cell is still alive. The Laser Therapy Lipolaser Plus slimming system can effectively remove excess fat without the negative side effects associated with surgical methods utilizing cold laser technology to emulsify fat and allow it to move to interstitial space.

Cold laser treatment alone helps patients reduce inches from their waist, hip and thighs with no other procedure involved. The patient must exercise for a half hour after the treatment to aide movement of the released fat. Cold laser therapy releases the cellular contents of fat cells while leaving the capillaries and other cells within the interstitial space intact.

The release of the fat cell contents is the result of the disruption of the cell's membrane. This membrane disruption, called a transitory pore, is the result of low-level laser stimulation. Prior to laser exposure, fat cells have a regular, cluster shape, as seen in Figure 1. Within minutes of low level laser exposure, fat cells begin to develop transitory pores as indicated by arrows in Figures 3-6. This pore allows the evacuation of the cell's contents into interstitial space (see Figure 4). Emptied fat cells (Figure. 7) can be passed through the body during its metabolism and normal course of detoxification.

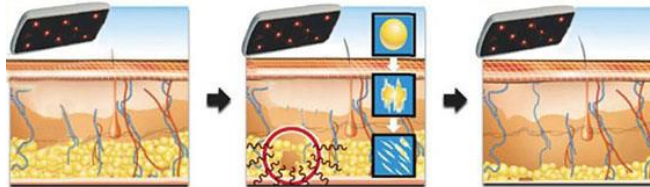

Lipo Laser Treatment Benefits

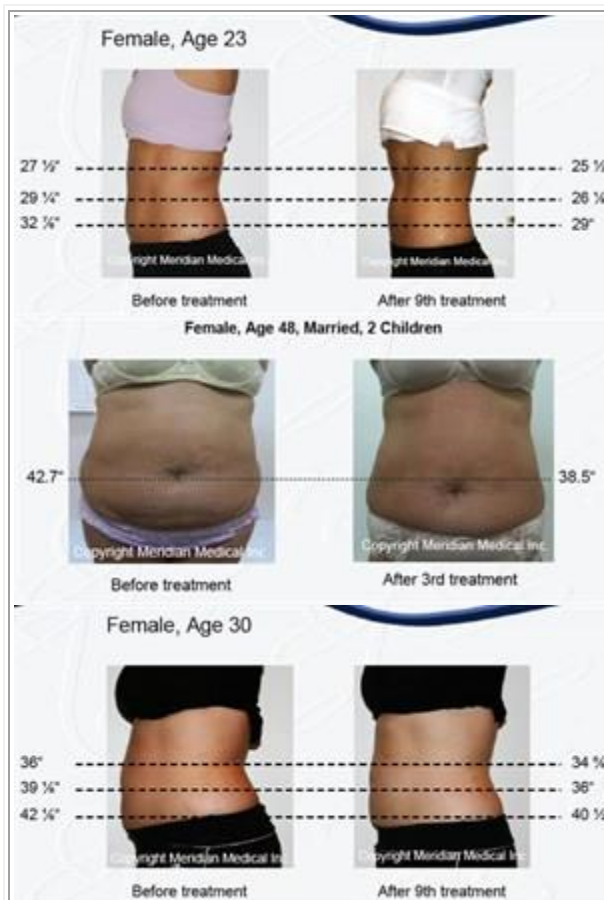

What can you expect?

You can relax in a comfortable position while a LipoLaser technician secures the multi-diode paddles and EN probes to the target area. In 10 minute intervals, you are free to relax, read and listen to music.

What happens after the treatment?

You are free to resume normal activities immediately after each session-including exercise. Remember the Lapex LipoLaser is safe, non-invasive and you will experience no pain or side effects.

How quickly will I see results?

Most patients will see measurable results after their first treatment. The standard treatment protocol consists of three 20 minute treatment sessions every week for 3 weeks. Results are enhanced by utilizing the Whole Body Vibration equipment for 10 minutes immediately after the treatment.

Who is the ideal candidate?

The LipoLaser is the ideal treatment for all body types when combined with a healthy diet and lifestyle. It works synergistically with the body's natural weight loss mechanisms. It specifically targets troublesome areas to remove inches.

Legal Disclaimers

The Laser Therapy Lipolaser Plus is a non-invasive low-level laser therapy device indicated for use in body contouring and temporary reduction in the appearance of cellulite. The device utilizes cold read laser for deep tissue warming to enhance micro-circulation for the reduction in the appearance of cellulite.

FDA approved Indications: The Laser Therapy Lipolaser Plus is intended to emit energy in the visible and infrared spectrum for the purpose of elevating tissue temperature for temporary relief of muscle and joint pain, stiffness, arthritis pain, Carpal Tunnel Syndrome; with temporary increase in local blood circulation.

This FDA cleared device for pain relief found a new off label use as a lipolaser. Much like other highly advertised Chiropractic Laser Treatment systems, the Laser Therapy Lipolaser Plus is a 100% Non invasive, laser-based, spot fat reduction and body contouring system using Cold Laser Lipo. This Laser Lipo treatment has the relaxation of a 40-minute massage with the benefits of inch loss! Rather than a painful incision, the Laser Therapeutics Lipolaser Plus uses laser energy to safely (and painlessly) using non invasive surgery to penetrate the skin and target specific adipose (or fat) cells. It is through this targeted process that the Laser Therapy Lipolaser Plus is able to in fact stimulate the cell itself into releasing water, free fatty acids, and glycerol without Laser Lipo Surgery. Having released these three compounds, water, free fatty acids, and glycerol (often referred to as Triglycerides), the fat cells "shrink" significantly, resulting in inch loss for patients!

